

## Aspirus Reference Laboratory - Specimen Transport Bags



**Clear:**  
Refrigerated Samples



**Blue:**  
Frozen Samples



**Green:**  
Room Temperature Samples



**Yellow:**  
Pathology Samples  
Goes to Assoc. in Pathology



**Red:**  
STAT Samples – Check FROZEN, REFRIG, or ROOM TEMP

## Packaging Reminders



- Place paperwork in ***the outside pocket***--located on the back of each bag-- to keep it separate from the specimen, in the event of leakage
- Place urine or stool samples in separate bags from blood samples
- Secure capped containers tightly, bag separately
- Seal the bag closed, with minimal air contained